

Assembly instructions for the *Caddy*



1. attach the two-part clamp on to the wheelchair frame as close to the wheels or wheel locks as possible
2. orient rod slightly "downhill" so that the Caddy wants to stay in the closed position
3. if you want to position the Caddy on the left side of the chair relocate the key holder on the opposite end of the extrusion

two-part clamp which attaches to the wheelchair frame

allen head bolt (tighten to position and hold clamp on to the frame)

plastic rod (attached to the clamp)

aluminum extrusion (attached to the Caddy)

stainless steel key holder (removable for right or left side use)

allen head screw (holds key holder on to extrusion and acts as a stop)

