

**!ATTENTION: READ BEFORE ASSEMBLING/OPERATING!**

**ASSEMBLY & PACKING INSTRUCTIONS FOR THE MULTICHAIR 5100tx**



**multichair 5100tx**

**!ATTENTION: READ BEFORE ASSEMBLING/OPERATING!**

**November 2016**

# ASSEMBLY INSTRUCTIONS

## !ATTENTION: PLEASE READ!

Please see Page 6: LIMITED WARRANTY of these instructions. If the warranty is not acceptable, contact Nuprodx, Inc. for information about returning the chair for a refund (Restocking fees and shipping charges will be applied). This chair is designed and intended for indoor shower and bathroom use. It is not designed for outdoor use and will void the warranty if used in this fashion. If thresholds, shower curbs or barriers are encountered that exceed ½" in height, the area in question should be brought within the code for disabled accessibility.



Figure 1

- A** The multiCHAIR 5100tx is shown in Figure 1.
- B** Unpack all parts from the shipping carton. Remove all tape and/or packing from the chair. Keep shipping materials and sheet-plastic which protects parts of your chair.
- C** Open the seat frame, lock the slide bolts, and position the frame upside down (Refer to Figures 2 & 3).
- D** Position the legs so that the brass buttons point outward (Making it easier for future leg height adjustment). Make sure that the legs slide "home" into the socket of the seat frame. Use the included ¼" Allen wrench to securely tighten the bolts located at the corners of the frame (Refer to Figure 4).

### SAFETY PRECAUTION!

If any of the legs are not fitted correctly into the socket of the seat frame or are not securely tightened there is an extreme risk of the legs dislodging completely from the chair. This could cause serious injury or even death!

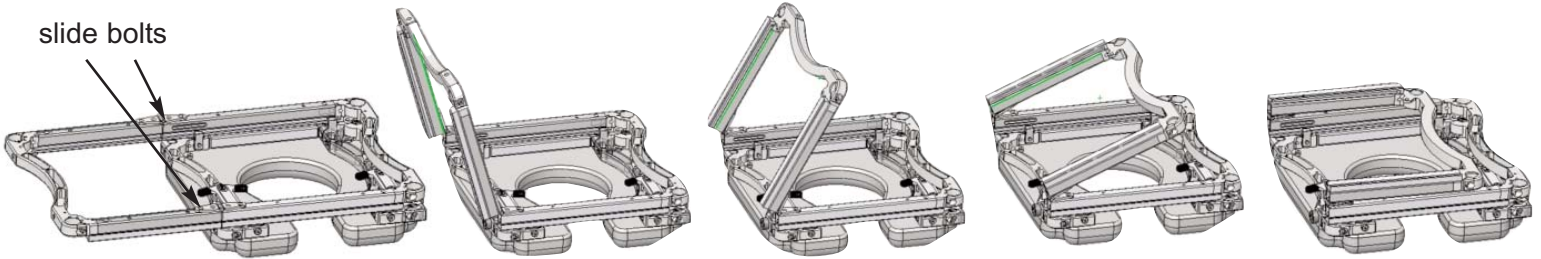


Figure 2

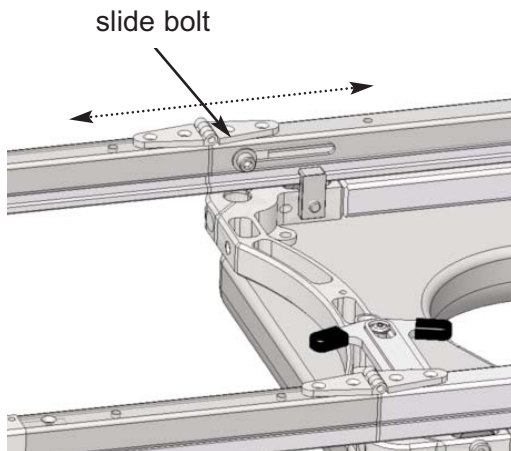


Figure 3

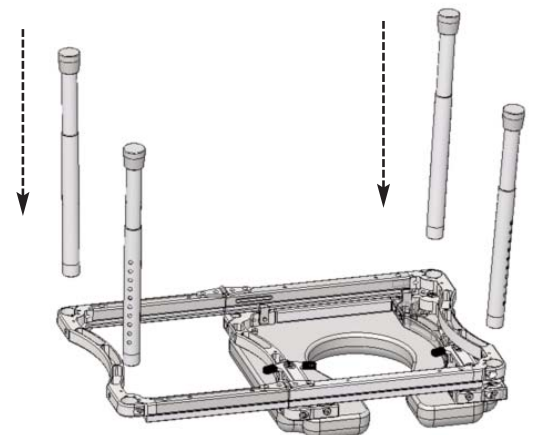


Figure 4



# OPERATING INSTRUCTIONS

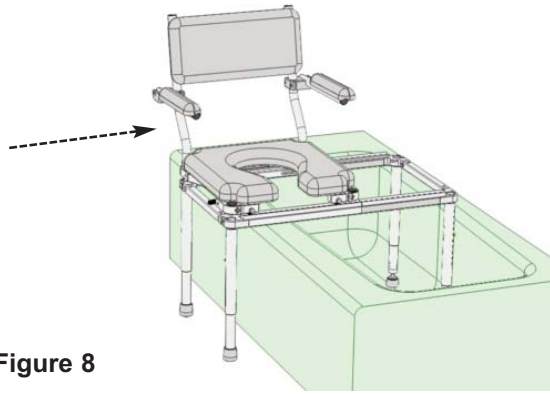


Figure 8

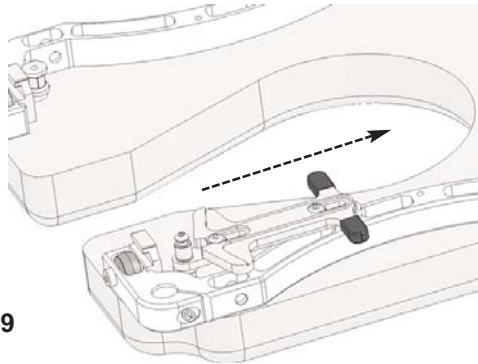


Figure 9

**A** Ensure all fasteners (bolts/washers) are securely tightened and all chair parts have been assembled correctly (Refer back to Assembly Instructions if needed. Contact NuprodX, Inc. if unsure of anything).

**B Commode Chair:** Position the multiCHAIR over the toilet, so that the center of the commode opening is directly above the center of the toilet bowl. Adjust the height of the multiCHAIR accordingly by setting the position of the brass buttons on the four inner legs to the various adjustment holes on the outer legs.

## SAFETY PRECAUTION!

Ensure that the height of each inner leg is set to make the multiCHAIR level. If this is not properly done, the multiCHAIR could tip to one side causing the user to fall. This could result in serious injury or even death!

**C Shower Chair:** Position the multiCHAIR inside the bathtub/shower. Adjust the height of the multiCHAIR accordingly by setting the position of the brass buttons on the four inner legs to the various adjustment holes on the outer legs (Refer to Figure 8). Pull the release latch open on the slider seat frame and transfer into the tub portion until the latch engages onto the opposite stop (Refer to Figure 9).

## ADDITIONAL SAFETY PRECAUTIONS

### !ATTENTION: PLEASE READ!

**WARNING!** Failure to account for the aforementioned and the following safety precautions can result in serious injury to the user or even death!

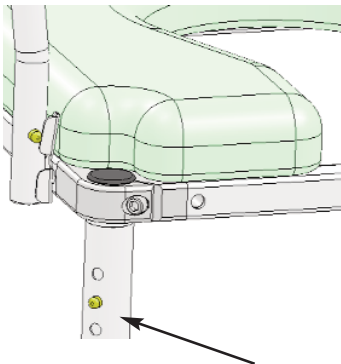


Figure 10

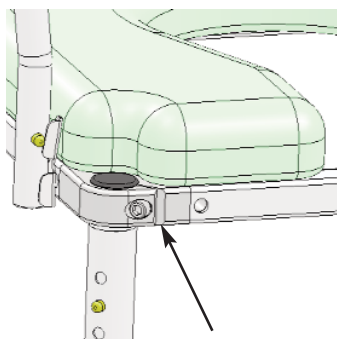


Figure 11

**A Leg Height Adjustment:** When making height adjustments to any of the legs, make sure that the brass buttons are fully engaged into the outer legs. If pushed too hard the button could become stuck inside the inner leg (Contact NuprodX, Inc. if this occurs). Leg height adjustment should only be made before transferring the user into the chair (Refer to Figure 10).

**B Crutch Tips:** Before transferring into and out of the chair, all four crutch tips must be securely planted onto the floor. Failure to do so could cause the chair to become unstable.

**C Fasteners:** Over time the screws and washers on the chair may become loose. It is very important that these are checked and securely tightened before each use. Failure to do so could cause a part to dislodge from the chair (Refer to Figure 11).



# PACKING INSTRUCTIONS



**Figure A** - Drape the black strap across the bottom of the carrying case (not shown for clarity). Place the inverted shuttle & track frame inside the case. Locate the plastic panel at the bottom of the frame.



**Figure B** - Place the legs in the inverted cushion and frame.



**Figure C** - Place the foam on top of the legs.



**Figure D** - Place the two side arms on top of the foam.



**Figure E** - Place a second piece of foam on the arms, and lay the back posts with rolled up cloth as shown.

**Figure F** - Tighten the cloth belt around the complete frame.