

!ATTENTION: READ BEFORE ASSEMBLING/OPERATING!

ASSEMBLY INSTRUCTIONS FOR THE NUPRODX MULTICHAIR 6000RS



!ATTENTION: READ BEFORE ASSEMBLING/OPERATING!

!ATTENTION: PLEASE READ!

Please see LIMITED WARRANTY on the last page of these instructions. If the warranty is not acceptable, contact Nuprodx, Inc. for information about returning the chair for a refund (restocking fees and shipping charges will be applied). This slider is designed and intended for indoor shower and bathroom use. It is not designed for outdoor use and will void the warranty if used in this fashion. If thresholds, shower curbs or barriers are encountered that exceed ½" in height, the area in question should be brought within the code for disabled accessibility.

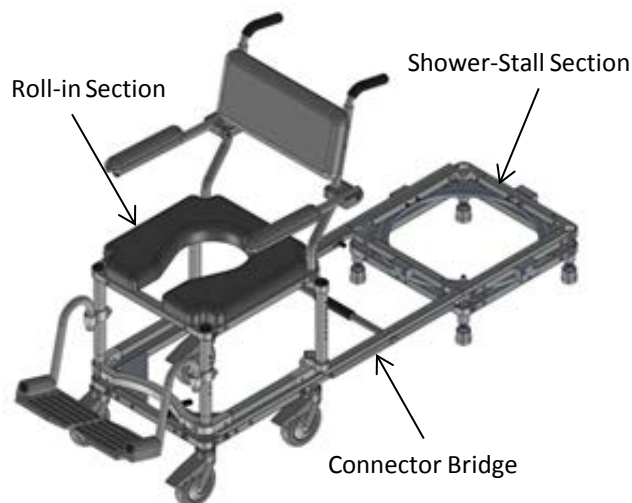


FIGURE 1

A. The multiCHAIR 6000RS (refer to Figure 1) is comprised of three sections: the roll-in section, the connector bridge section and the shower-stall section. Unpack all items from the shipping carton and layout.

B. Remove all tape and/or packaging from the parts. Keep shipping materials for future use and note their locations.

C. Position the frames per Figure 2 and install the casters onto the roll-in section frame and the adjustable feet onto the shower-stall section. The casters and adjustable feet are installed by sliding their 1.25" diameter plugs into the mating bored holes on the frames. Make sure they slide all the way in and then tighten the bolts at the associated frame corners with the ¼" Yellow T-Handle Allen wrench provided. The connector bridge is then set into place by setting the (4) coned ends into the mating eyelet features on the frame corners. There are locking slide bars at the (4) ends of the bridge that slide to lock the bridge to both frames.



FIGURE 2

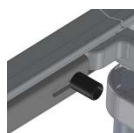


FIGURE 3

SAFETY PRECAUTION!

All caster inserts and shower section leg inserts need to be properly fitted into their respective frame sockets and the associated pinch clamp bolts need to be securely tightened. Failure to do this creates a risk which could result in serious injury or even death!

The connector bridge must be fully seated into the corresponding holes of both the roll-in section and tub section frames. If this is not done correctly, the shuttle/seat frame could dislodge itself from the base frames when transferring from the roll-in section to the shower-stall section. This could cause serious injury or even death!



FIGURE 4

D. Install the legs into the shuttle frame by sliding the leg ends into the 1" diameter holes on the shuttle frame corners and positioned per Figure 4. Again, make sure that the legs go all the way in and then tighten the "pinch-clamp" corners with the ¼" Yellow T-Handle Allen wrench.



FIGURE 5

E. Install the seat frame by lining up the leg holes at the bottom of the (4) corners of the seat frame with the legs per Figure 5. Slide the seat frame down onto the legs until they "bottom-out". Once the legs have bottomed out and are securely fitted into the holes. Tighten the associated "pinch-clamp" corners with the ¼" Yellow T-Handle Allen wrench.

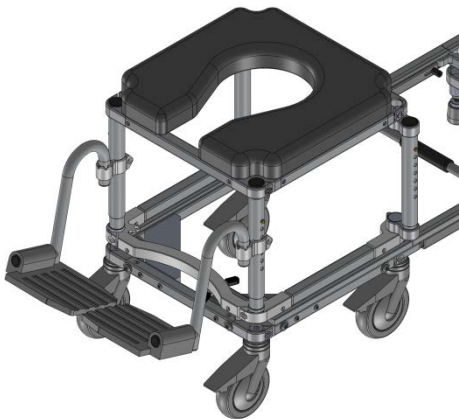


FIGURE 6

F. Place the footrests in footrest clamps per Figure 6. The footrests are height adjustable by positioning the footrest clamps along the front legs and again, using the ¼" Yellow T-Handle Allen wrench, tighten in the desired positions.



FIGURE 7

G. Install the back assembly by sliding the (2) posts of the back rest assembly into the rear seat frame holes, see Figure 7 for proper positioning. The back posts are then secured in place by tightening the internal back posts clamps with the ¼" Yellow T-Handle Allen wrench. See Figure 8 for positioning of the back posts sliding into the rear seat frame holes and the bolts that need to be tightened once the posts are slid down into place.

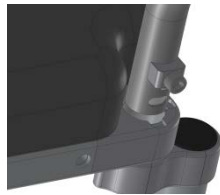


FIGURE 8

H. See Figure 9 for the positioning of the cantilever arms on the back posts. The height of the arms relative to the seat and back cushion can be adjusted to properly fit the user. The arms are mounted to the back posts with a hinged clamp. Initially remove the bolts from the hinged cantilever arm clamps and wrap the clamp around the back post, reinstall bolts and start to tighten with the ¼" Yellow T-Handle Allen wrench, see Figure 10 for position of hinged clamp bolt and mounting location along back post. Once in position, securely tighten the arm clamp bolts. At this point, your mc6000RS should be completely assembled.



FIGURE 9



FIGURE 10

SAFETY PRECAUTION!

Now that your mc6000RS is fully assembled, go back and ensure that all bolts are properly tightened and that everything fits together properly. Failure to do so could result in parts coming loose and could cause serious injury or even death!

OPERATING INSTRUCTIONS

I. Now that the unit is assembled, remove the connector bridge and place the shower-stall section in the shower stall to be used, see Figure 11. Connect the two frames with the connector bridge and “lock” the bridge in place with its (4) slide bars. Then you need to set the tracks of both frames at approximately the same height and be level. This is accomplished by screwing the (4) adjustable feet on the shower-stall section, see Figure 12. Once the shower-stall section height is correct, “lock” the leg heights by screwing the jam nuts up against the insert plugs. A 7/8” open ended wrench or a small crescent wrench can be used to do this.

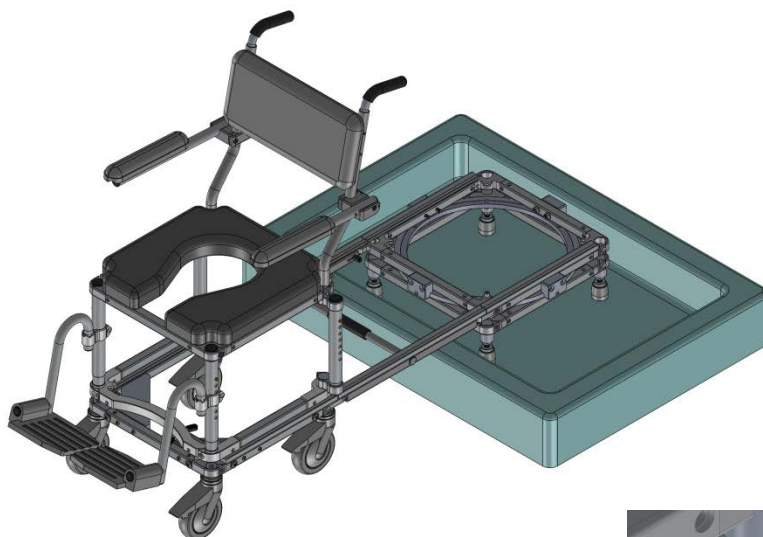


FIGURE 11

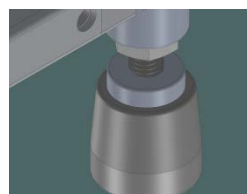


FIGURE 12

J.) The shuttle section is now ready to slide back onto the shower-stall section across the connector bridge. This is done by pulling back on the slider latch at the front base of the shuttle and simultaneously pushing the shuttle frame towards the shower-stall section, see Figures 13 and 14.

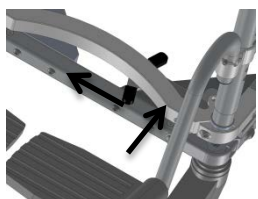


FIGURE 13

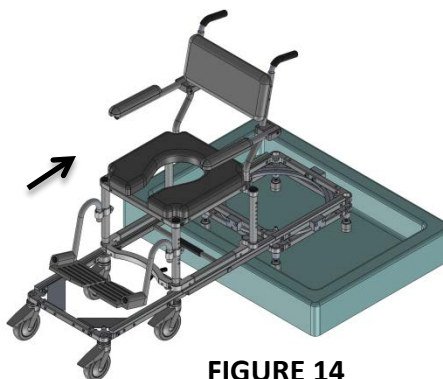


FIGURE 14

K. Once the shuttle frame is slide all the way across onto the shower-stall section, the latch will engage with a mating “stop” on the shower-stall section. The connector bridge can then be removed the shower-stall section can then be rotated to align the user up in front of the shower head, see Figure 15.

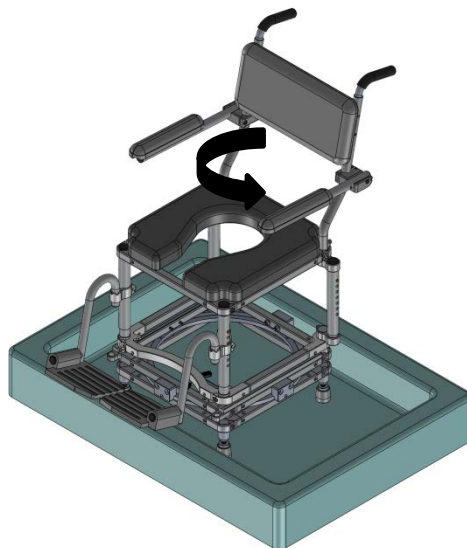


FIGURE 15

L. Reverse the steps to get the shuttle frame slide out onto the roll-in section once the user has completed their shower. The roll-in section can be used directly over a commode as well, see Figure 16, either before or after showering.



FIGURE 16

QUESTIONS?

For assembly or operating questions, please call Nuprodx, Inc. at 415-472-1699 or email info@nuprodx.com.

LIMITED WARRANTY

A WARRANTY ACTIVATION Please read this warranty before operating or using your multiCHAIR. To activate the warranty for your multiCHAIR, register it online (www.nuprodx.com/form_registration/index.htm) or by phone. By operating or using the chair, you agree to the terms of this warranty.

B WARRANTY: Nuprodx, Inc. warrants this product against defects in material and workmanship as follows: There is a *10-Day conditional money-back warranty*. During this time period, the customer is permitted to try out the multiCHAIR, fully-clothed, keeping the chair in "like new" condition (NOTE: "Like new" condition, in terms of the multiCHAIR, is defined by Nuprodx, Inc. as no visible wear/usage/water marks for the multiCHAIR to be accepted and refunds issued when returned to Nuprodx, Inc.). If the customer decides that the multiCHAIR will not work for them, before the 10-Day period has passed and with the authorization of Nuprodx, the chair can be returned for a full refund minus a *10% restocking charge* (NOTE: The customer is responsible for both in-bound and out-bound freight). After the initial 10-Day period has passed, there is a *two-year limited warranty* for all parts of the chair, with the exception of the seat and back cushions (NOTE: Because of the fragile nature of the foam, there is no warranty for the cushions). The warranty does not cover normal "wear and tear" from everyday use of the product and custom parts/custom cushions are also excluded from the warranty and cannot be returned for a refund under any circumstances. Please see section E, WARRANTY LIMITATION AND EXCLUSIONS for more info on warranty exclusions.

C The warranty period begins on the date you receive the chair. For warranty service, please contact Nuprodx, Inc. no later than one month following the applicable warranty term. The chair will be repaired or replaced at the discretion of Nuprodx, Inc. with no charges to you for parts and labor, provided you have proof of purchase and of purchase date.

D DISCLAIMER: Except for the above warranty, and the acknowledgement by Nuprodx, Inc. that the chair, as manufactured by it, is fit for the general purpose for which most persons acquire a chair of its kind, Nuprodx, Inc. provides that you accept the chair as is, without warranties, either express or implied. Nuprodx, Inc. makes no warranty of fitness for your particular purpose and no warranty of merchantability beyond that already stated. No warranties extend beyond the duration of the express warranty stated above.

E WARRANTY LIMITATIONS AND EXCLUSIONS: The only obligation of Nuprodx, Inc. is to provide the purchaser with free repair and replacement as described above. This exclusive warranty remedy will not have failed as long as Nuprodx, Inc. is willing and able to repair or replace as described, but if this remedy should be held to have failed, the only remaining warranty obligation of Nuprodx, Inc. shall be to refund the acts beyond the control of Nuprodx, Inc. The warranty does not cover normal "wear and tear" from everyday use of the product. Standard seat/back cushions/custom parts/custom cushions are not covered under the warranty.

F This warranty gives you specific legal rights, and you may have other rights that may vary from state to state.

G This warranty does not apply to problems arising from normal wear, improper operation, improper maintenance, improper storage or similar disclaimer of implied warranties, and some do not allow limitations on how long an implied warranty may last. Some do not allow exclusion or limitation of incidental or consequential damages. So the above limitations or exclusions may not apply to you.

H RETURN INSTRUCTIONS In the event that you need to return the Nuprodx multiCHAIR, please follow the instructions below:

- 1) Review the information contained in your warranty to see if this applies to you
- 2) Obtain a Return Authorization # from Nuprodx, Inc.
- 3) Re-package the entire chair and its contents in the original packaging and ship to the following address:

Nuprodx, Inc.
Attn: RMA# XXXXXXX
161 S. Vasco Rd.; Suite G
Livermore, CA 94551